



USER MANUAL

Taper Pin

Version 0 – Last version date of the manual 04-11-2024

Stiletto Representative for EU Compliance

Krzysztof Bogucki
Przemysłowa 64/BOX 2
61-541 Poznań
+48 789-658-580

E-mail: krisuprzebudzenie@gmail.com; Website: www.stilettosupply.com

If in need of technical assistance with the product, contact the manufacturer at the addresses indicated above.

General presentation

Before carrying out any operation with the product in question, ensure you have read the entire manual.

Stiletto declares that the information contained in this document is congruent with the technical and safety specifications of the product to which the manual refers.

A QR Code leading to this safety manual will be enclosed within the packaging of each product.

Stiletto does not recognize any documentation that has not been produced, released, or distributed by itself or by its authorized representative. This manual will be accessible online for an unspecified period of time and may be viewed by anyone. After this period, it will be the obligation and responsibility of those who manage the product to ensure that both the product and the documentation comply with the laws in play at the time of the inspection.

Warnings

This manual describes the general safety rules to be observed during any operation carried out with the taper pin. The procedures described in the following paragraphs must be carried out with respect to both the indicated execution methods and the general safety regulations. Different countries may have different regulations regarding safety; in the case that the rules of the manuals conflict with the rules of the country in which the product is sold, the rules of the country will have priority over those of the manuals.

- When obtained, check the integrity of the product and its packaging to ensure it has remained in a sterile state. If you find any damage or trace of impacts due to transport, notify Stiletto representative Federica Mascetti before proceeding with the subsequent operations. **NEVER USE DAMAGED PRODUCTS.**
- **Before use, check the expiration date of the product.**
- Keep out of reach of children and animals. This product should only be used by adults.
- Never use product in the mouth or nose for purposes other than those for which it is designed and used, i.e. the creation of piercing holes.

Declaration of conformity

The declaration of conformity made in accordance with legislation in the European territory is also included with the sale of this product.

**NOTE: BEFORE USING THE PRODUCT IN ANY FORM, CHECK FOR THE DECLARATION OF COMPLIANCE;
IF THE PRODUCT IS SOLD TO A THIRD PARTY, ALL DOCUMENTATION MUST BE DELIVERED WITH IT.**

Product Description

A steel cone made for inserting jewels that has a blunt tip at the top and a rounded back for holding body piercing jewelry and internal jewelry. This product is presterilized, disposable, and available in different shapes and sizes. Taper pins are individually packaged and sold in packs of several pieces. The product is neither a toy nor a medical device.

The product is intended to be used for aesthetic treatments by professional body piercers for the purpose of making piercings.

Technical data

Physical characteristics	
Material:	304S Steel coated with medical silicone oil
Sterilization method:	Ethylene oxide
Packaging:	<ul style="list-style-type: none">• Single taper pin: packaged in Kraft paper package.• Box: Cardboard.
Sterility validity of the intact package	5 years. Each product shows the expiration date by which it must be used on the individual paper packaging.
Dimensions (cm)	6.5 x 2.5 x 8.5
Weight (oz)	½ Length: 1.1, 18g: 1.6, 16g: 2.3, 14g: 1.7, 12g: 2.3

Residual Risks

The manufacturing company has carried out an in-depth risk analysis of the product in question. From this analysis, risks emerged that cannot be eliminated due to their nature, and the recommendations for how to avoid them were emphasized in the following manual. As such, anyone who must use and maintain the product must read the manual beforehand.

- Store this manual carefully as it is necessary for correct and safe use of the product.
- Always check the condition of the packaging and the expiration date before using a product.
- If the structure of the product presents deformations following an accidental impact that could presume it to be dangerous and/or unusable, replace the product.
- The product is intended to be used only by professional body piercers for making piercings.
- Any product updates are published at the following link: www.stilettosupply.com

Conservation

Before storing the product, check the condition of the packaging to ensure there are no traces of impacts, abrasions and/or traces of dirt, rust, mold, etc. which may have been caused by incorrect transport and/or storage. If so, notify the manufacturing company.

- Store the product inside a room-temperature, dry environment away from heat and sunlight.
- Product should remain inside its original packaging.

Handling and unpacking

Remove the taper pin delicately from the pouch so as not to injure yourself or damage the product; dispose of the packaging according to local regulations. Product should only be removed from the sterile pouch immediately before use.

Usage



The product is intended only for adults qualified as body piercers and should be kept away from children and animals.

The product is disposable. Do not use the product if the packaging is damaged before use.

NEVER use beyond the expiration date.

Usage procedure

- Sanitize your hands by washing them abundantly with soap and water.
- Wear protective gloves before handling the instrument and its packaging.
- To remove the taper pin, grasp the two unsealed edges of the short side of the pouch. Pull the two flaps in opposite directions so that the welding on the sides opens up to approximately ¾ of its length.
- Grasp product carefully with your hand by the long section.
- Place the back of the cone into the internal thread of the jewelry.
- Proceed with the aesthetic treatment.
- Remove the cone from the jewel.
- Dispose of the product safely.

THE MANUFACTURER CANNOT BE HELD RESPONSIBLE IN ANY EVENT FOR ACCIDENTS OR DAMAGES RESULTING FROM THE INAPPROPRIATE USE OF THE PRODUCT, AS WELL AS FOR EVEN PARTIAL FAILURE TO OBSERVE THE SAFETY RULES AND INTERVENTION PROCEDURES DESCRIBED IN THE DOCUMENT.

Disposal

The packaging and cardboard box must be disposed of in separate waste according to current regulations as specified on the environmental label.

The product must be disposed of within the guidelines for biohazard disposal according to your area.



Warranty

The warranty provisions listed in full in the purchase contract are only valid if the product is used for the intended purpose. The warranty does not extend to damage caused by incompetence, negligent use of the product, or poor conservation.

USE: FAILURE TO COMPLY WITH MODE THE USE OF THE PRODUCT DESCRIBED IN THIS DOCUMENTATION RESULTS IN THE VOIDING OF THE WARRANTY TERMS

The products are covered by warranty regarding the following conditions:	
1	<i>The guarantee is valid for a period of 12 or 24 months depending on the type of buyer.</i>
2	<i>The manufacturing company undertakes to replace incorrectly manufactured parts at its discretion after careful inspection and verification of poor construction.</i>
3	<i>The buyer is always responsible for transport and/or shipping costs in the event of incorrect use of the guarantee.</i>
4	<i>Only the original purchaser who has complied with the instructions for normal use, cleaning, and storage contained in the manual can benefit from this guarantee. Our warranty liability expires when the original owner relinquishes ownership of the product or when modifications are made to it.</i>
5	<i>The warranty does not include damage resulting from excessive stress such as using the product after detecting an anomaly, the use of inappropriate operating methods, or failure to follow the instructions for use.</i>
6	<i>The manufacturer assumes no responsibility for any difficulties that may arise in resale or use abroad due to the provisions in the country in which the product was sold.</i>

If it is deemed necessary to use the guarantee, please indicate the following data:

1	<i>Name of product</i>
2	<i>Purchase date (must include documentation for purchase presentation)</i>
3	<i>Detailed description of the problem</i>

USE: FAILURE TO COMPLY WITH THE USE OF THE PRODUCT DESCRIBED IN THIS DOCUMENTATION RESULTS IN THE VOIDING OF THE WARRANTY TERMS.

